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009495EX-20180227

(EN) Instructions for use breCAM.multiCOM

Before using the product, please read these instruction for carefully use.

The manufacturer will not accept any liability for damage resulting from non-compliance with these instructions for use.

1. Description of the product

breCAM.multiCOM is based on a polychrome coloured, crosslinked polymethyl methacrylate (PMMA).

breCAM.multiCOM is suitable for the manufacture of dental prostheses in the form of crowns and bridges using CAD/CAM techniques.

2. Indication

breCAM.multiCOM is used for the fabrication of crown and bridge restorations with a maximum period of wearing of 2 years using CAD/CAM techniques.

3. Contraindication

If intolerance is suspected, this product must only be used once allergy investigations have taken place and there is proof that no allergy exists.

4. Safety recommendations and hazard warnings

4.1 CAM – Processing

Milling chips should be collected during dry processing in the CAM system. When using unsuitable milling tools, bur fracture may occur. When overheating the milling material, the milled unit (restoration) may be deformed and destroyed.

4.2 Labor – Processing

The standard precautionary measures need to be taken when processing the material. A respiratory mask and an extrac-tion system are required to avoid the inhalation of dust. For personal protection, safety goggles should be worn to avoid damage to the eyes caused by foreign particles.

To ensure the quality and reliability of the breCAM.multiCOM material, all the information and notes need to be observed.

5. Storage and durability

The product is stored in the original bredent breCAM.multiCOM packaging at normal room temperature.

The product should be kept out of sunlight and stored in a dry place.

6. Processing and use Design

The minimum wall thickness of the structures should be 0.6 mm (cervical) and > 1,0 mm (occlusal). Cross-sections of connectors should be at least 10 mm² in the anterior region and 15 mm² in the posterior region. Restorations should not have more than one pontic between two abutments.

CAM processing

breCAM.multiCOM can only be used in milling systems without radio frequency identification (RFID).

We recommend a speed of approx. 19,000 rpm at a feed rate of 18 mm/s for a milling tool with a diameter of 2 mm. The feed depth should be 0.5 mm. We recommend a speed of approx. 24,000 rpm at a feed rate of up to 22 mm/s for a milling tool with a diameter of 1 mm.

Cutting/milling speeds must be observed. Generally, the user is responsible to adjust his milling system to the material requirements.

Bonding/cementing

breCAM.multiCOM is inserted using standard bonding/cementing materials. If adhesive bonding is intended, it must be ensured that the manufacturer’s instructions concerning bonding and use are adhered to.

7. Technical data (DIN EN ISO 10477)

E-modulus	> 2200 MPa
Flexural strength	> 100 MPa
Water solubility	< 0,8 µg/mm³
Water absorption	< 20 µg/mm³

8. Symbols

 European conformity marking

 Catalogue number

 Batch code

 Keep dry

 Keep away from sunlight

 Caution

 Manufacturer

9. Additional information

The product may only be used by dentists, dental technicians and adequately trained dental staff.

Only original tools and components may be used for processing.

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing.

Any occurring claims for damages may only be made up to the value of our products.

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bredent GmbH & Co. KG